EMTONE[®]

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EMTONE

ESTU

EMTONE™: SYNERGY OF SIMULTANEOUS DELIVERY OF RF AND TARGETED PRESSURE ENERGY FOR CELLULITE REDUCTION

SAFETY AND EFFICACY OF NOVEL TECHNOLOGY FOR CELLULITE TREATMENT BASED ON SIMULTANEOUS APPLICATION OF MONOPOLAR RADIOFREQUENCY AND TARGETED PRESSURE ENERGY

Mariano Busso, MD¹; Trevor M. Born, MD²

¹ Aesthetic Dermatology, Coconut Grove, FL, USA
² TMB Cosmetic Surgery, Toronto, ON, Canada

Presented at the Annual Meeting of the American Society for Laser Medicine and Surgery, 2020.

HIGHLIGHTS

- 30 subjects with cellulite (grade 2 to 3) were enrolled. All subjects underwent four treatments (two sessions per week) in desired area.
- 87% of the subjects reported improvement in the appearance of the treatment area.
- Total of 86% of the photographs were rated as significant improvement in the treated area.
- 90% of the patients would undergo the EMTONE treatment again.

Example of patient photographs before (left) and after 4th treatment.

KEYWORDS

cellulite, noninvasive, radiofrequency, laser

This is an open access article under the terms of the Creative Commons Attribution-NonCommercial-NoDerivs License, which permits use, distribution and reproduction in any medium, provided the original work is properly cited, the use is non-commercial and no modifications or adaptations are made.
© 2018 The Authors. Journal of Cosmetic Dermatology Published by Wiley Periodicals, Inc.
J Cosmet Dermatol. 2018;1-4.

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
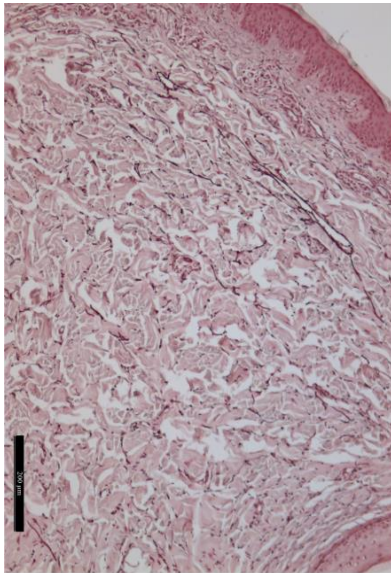
HISTOLOGICAL STUDY: SIMULTANEOUS vs. CONSECUTIVE APPLICATION

HISTOLOGICAL EXAMINATION OF SKIN TISSUE IN THE PORCINE ANIMAL MODEL AFTER SIMULTANEOUS AND CONSECUTIVE APPLICATION OF MONOPOLAR RADIOFREQUENCY AND TARGETED PRESSURE ENERGY. Brian Kinney M.D., Dian Kanakov M.D., Penka Yonkova, PhD

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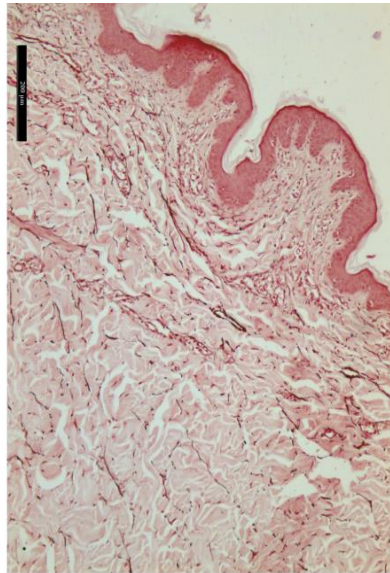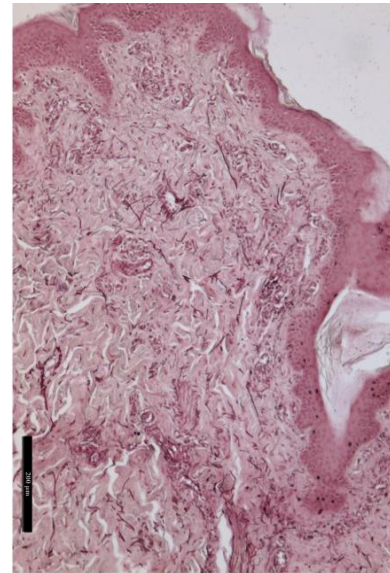
BTLAESTHETICS.COM

HISTOLOGIA – COLÁGENO e ELASTINA


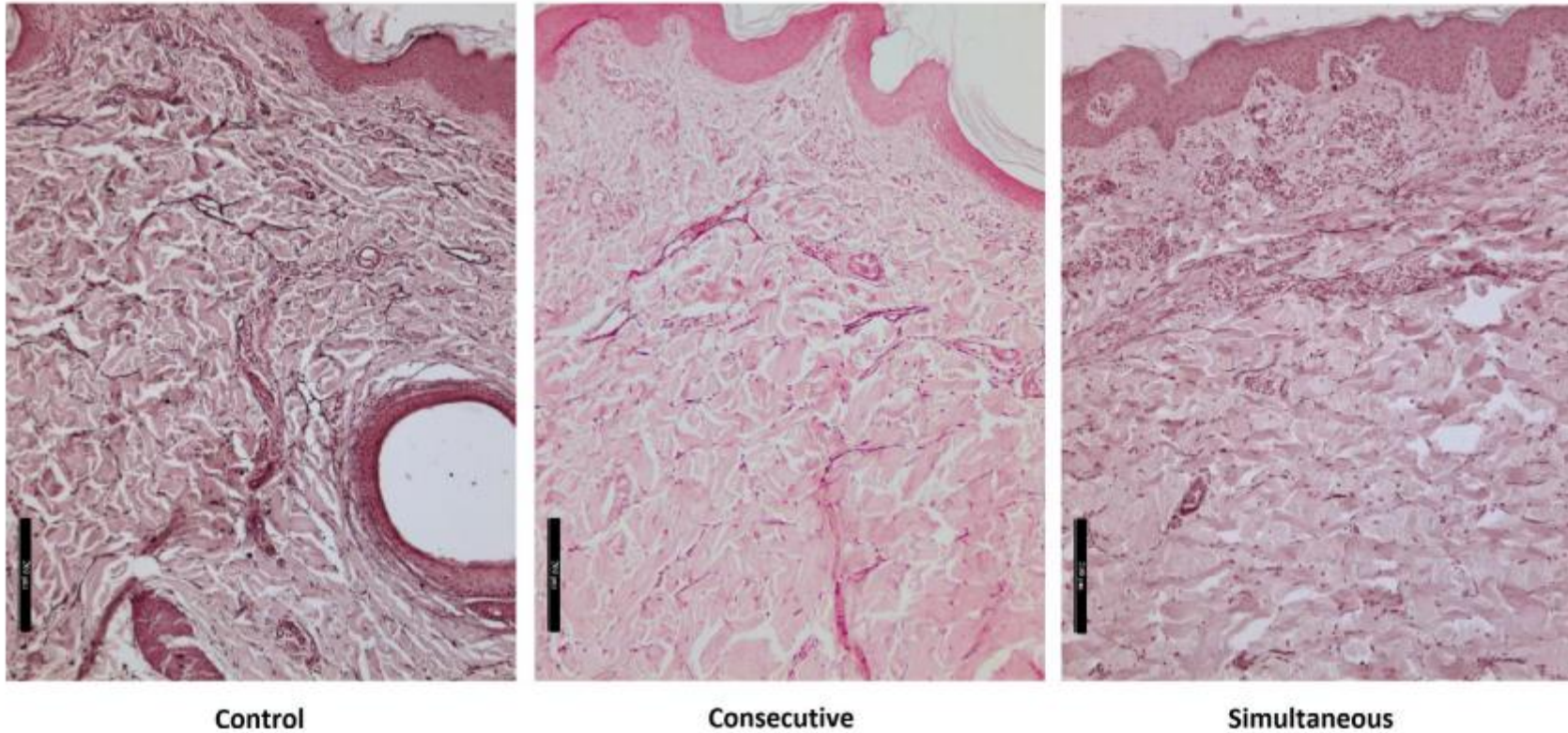
FIGURE 2 Increase of collagen and elastin amount—area encompassed on examined slices (A, mean \pm SD). B, shows illustrative histological comparison of control/consecutive/simultaneous samples at 1-mo follow-up. The collagen and elastin fibers are denser and thicker in the consecutive/simultaneous samples. Samples of treated tissue are also showing greater vascularization, especially in the upper part of reticular dermis. Detail of the dermis, bar 200 μ m

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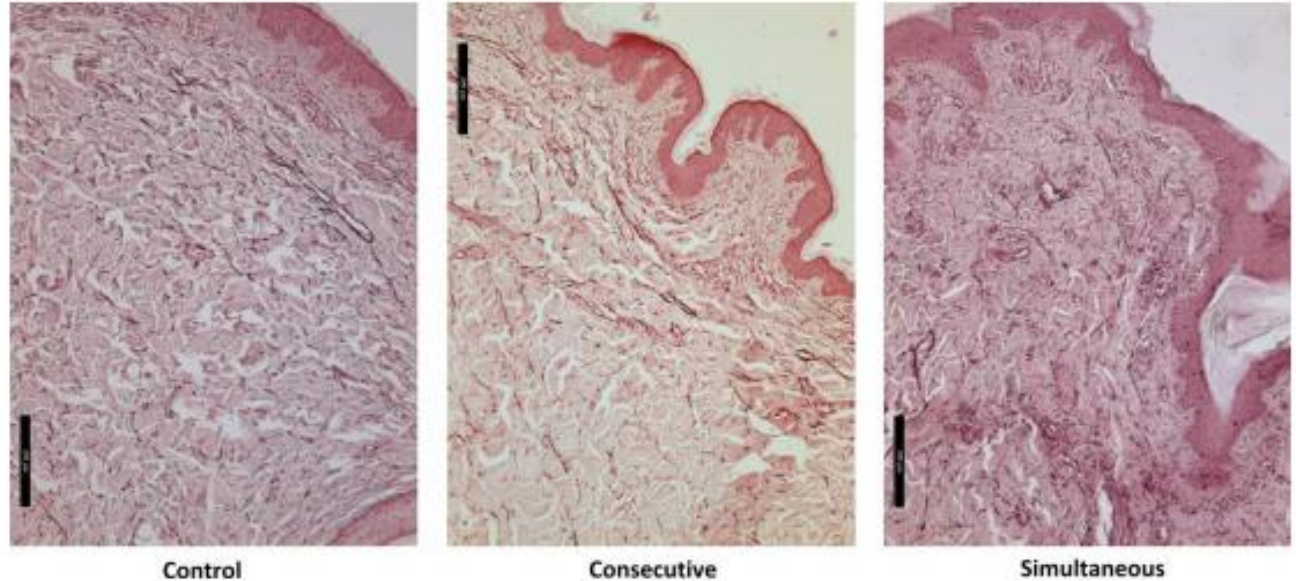FIGURE 6 Illustration of increased vascularization. Samples were taken at 1-mo follow-up. Consecutive/simultaneous samples are showing greater vascularization, denser organization, and thickness of connective fibers in comparison with the control. Detail of the dermis; bar 200 μ m

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CLINICAL EVALUATION OF SIMULTANEOUSLY APPLIED MONOPOLAR RADIOFREQUENCY AND TARGETED PRESSURE ENERGY AS A NEW METHOD FOR NON-INVASIVE TREATMENT OF CELLULITE IN POST-PUBERTAL WOMEN. Prof. Klaus Fritz M.D., Prof. Carmen Salavastru, M.D., Magdalena Gyurova M.D.

- 30 patients, 4 sessões, 3meses follow-up.

93%

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2.2 cm

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ULTRASSONOGRAFIA – espessura da derme, hipoderme

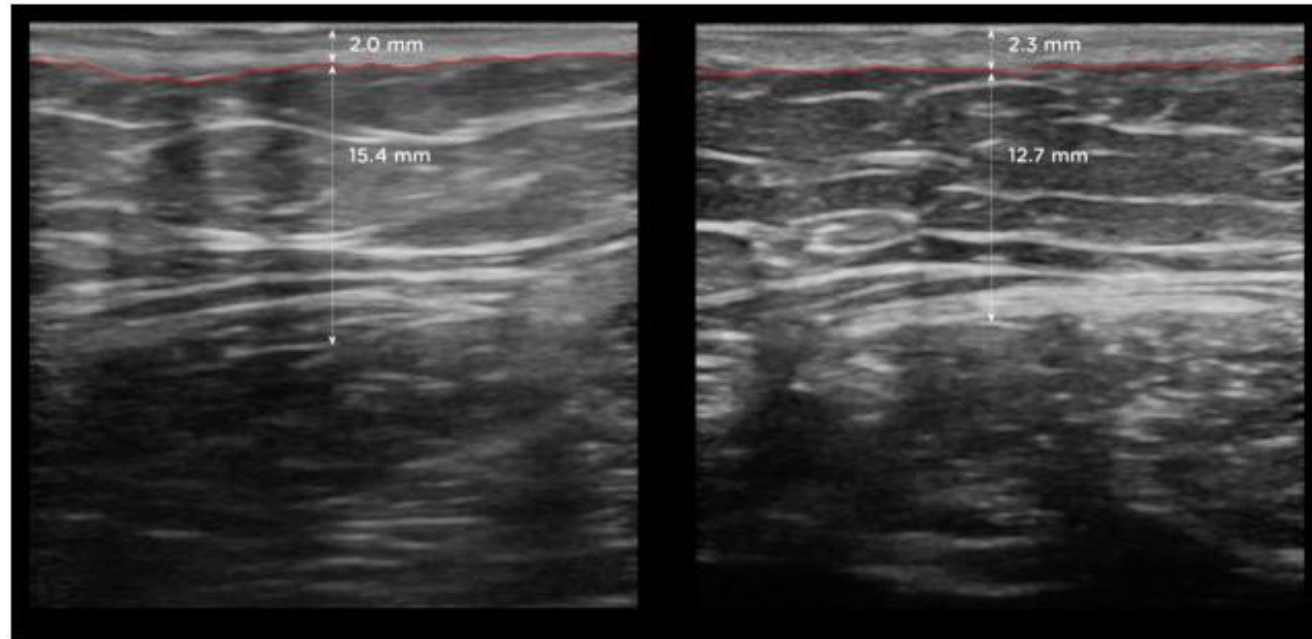
FIGURE 1 Ultrasonography showing changes in dermal and subdermal tissues before (left) and 3 months posttreatment (right)

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TERMOGRAFIA – homogeneidade do suporte sanguíneo

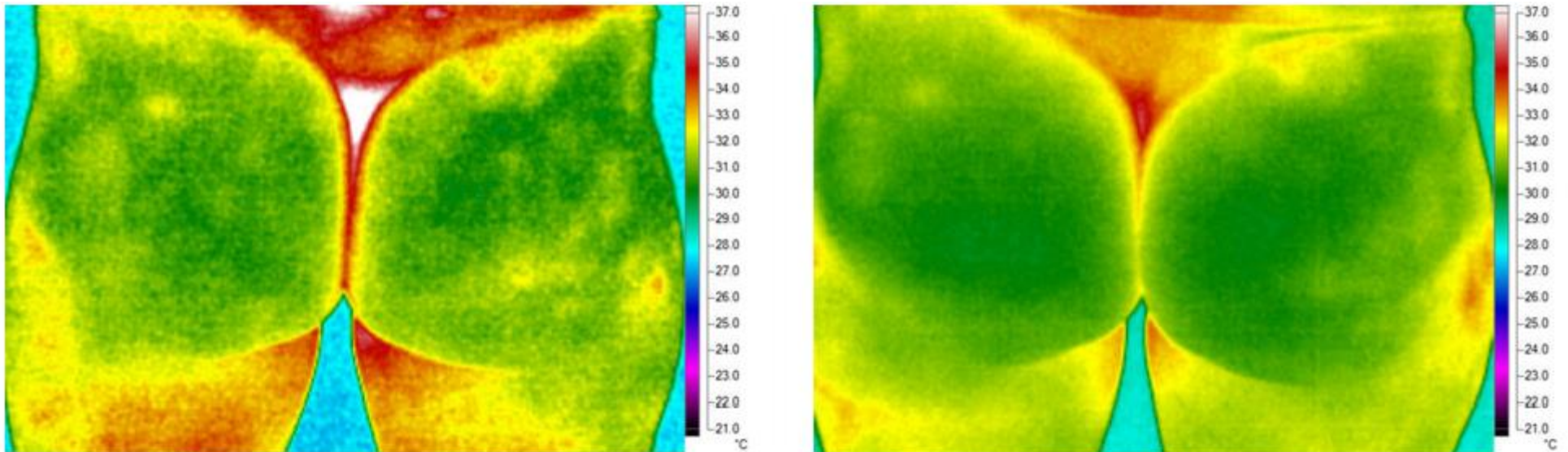
FIGURE 2 Thermography showing changes in the thermal profile before and 3 months posttreatment

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TABLE 2 Circumferential measurements and ultrasonography results

Measurement	Baseline	Follow-up	Difference	P-value
Hip circumference (cm; n = 30)	105.3 ± 8.4	103.0 ± 8.6	-2.3 ± 1.6	<.001
Thigh circumference (cm; n = 30)	63.9 ± 6.9	61.8 ± 6.9	-2.1 ± 1.7	<.001
Thickness of dermis (mm; n = 5)	2.0 ± 0.2	2.3 ± 0.2	0.3 ± 0.1	<.05
Thickness of subcutaneous tissue (mm; n = 5)	21.5 ± 2.7	19.5 ± 4.7	-2.0 ± 1.6	<.05

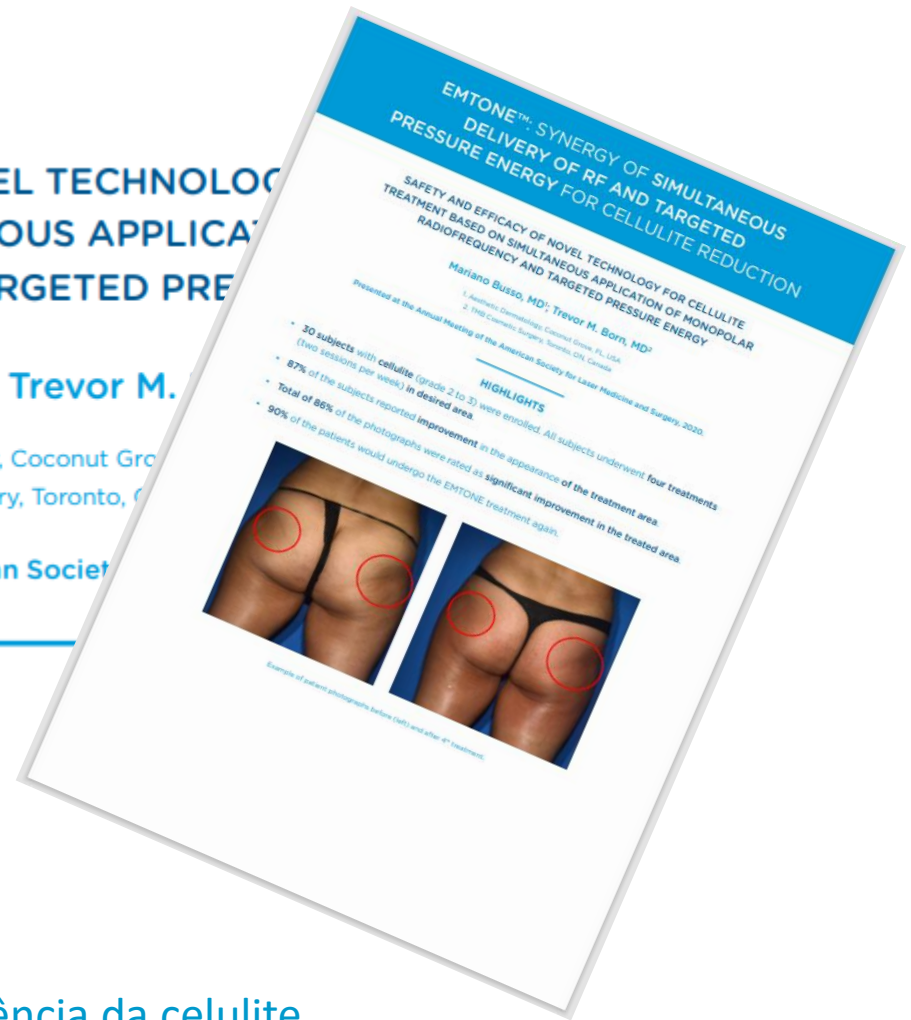
2.2 cm

SAFETY AND EFFICACY OF NOVEL TECHNOLOGY TREATMENT BASED ON SIMULTANEOUS APPLICATION OF RADIOFREQUENCY AND TARGETED PRESSURE ENERGY

Mariano Busso, MD¹; Trevor M. Born, MD²

1. Aesthetic Dermatology, Coconut Grove, FL, USA
2. TMB Cosmetic Surgery, Toronto, ON, Canada

Presented at the Annual Meeting of the American Society for Laser Medicine and Surgery, 2020.

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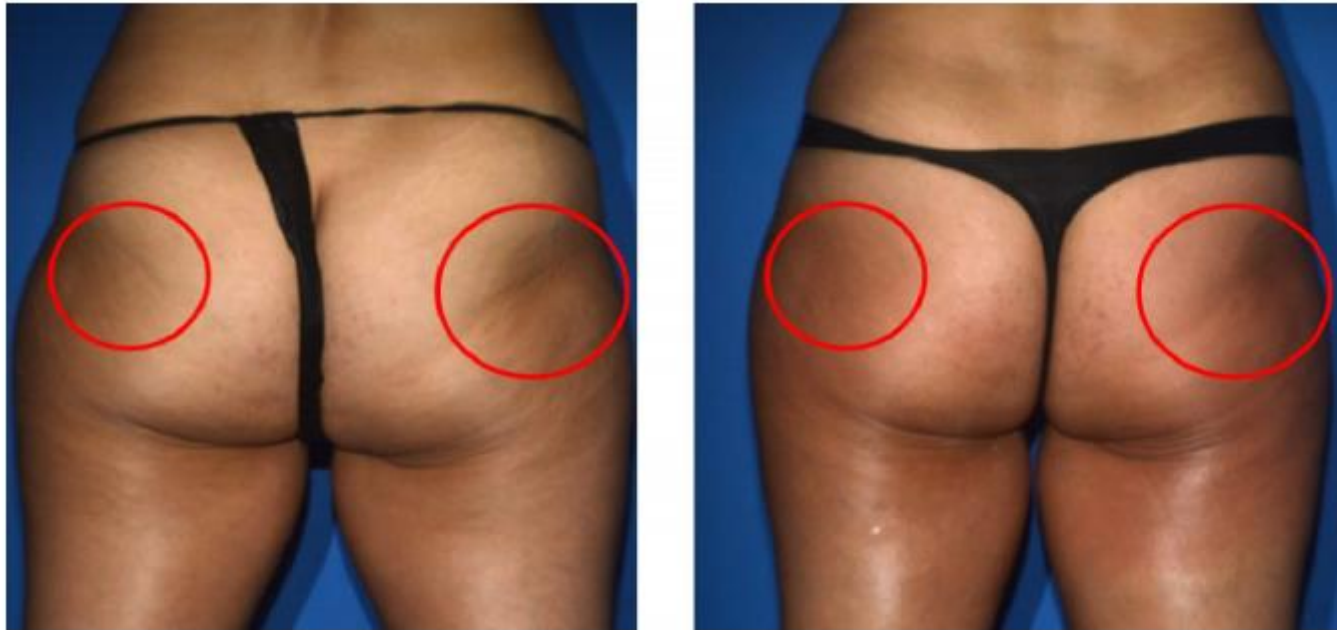
90% pacientes repetiriam o tratamento com EMTONE.

SAFETY AND EFFICACY OF NOVEL TECHNOLOGY FOR CELLULITE
TREATMENT BASED ON SIMULTANEOUS APPLICATION OF MONOPOLAR
RADIOFREQUENCY AND TARGETED PRESSURE ENERGY

Mariano Busso, MD¹; Trevor M. Born, MD²

Example of patient photographs before (left) and after 4th treatment.

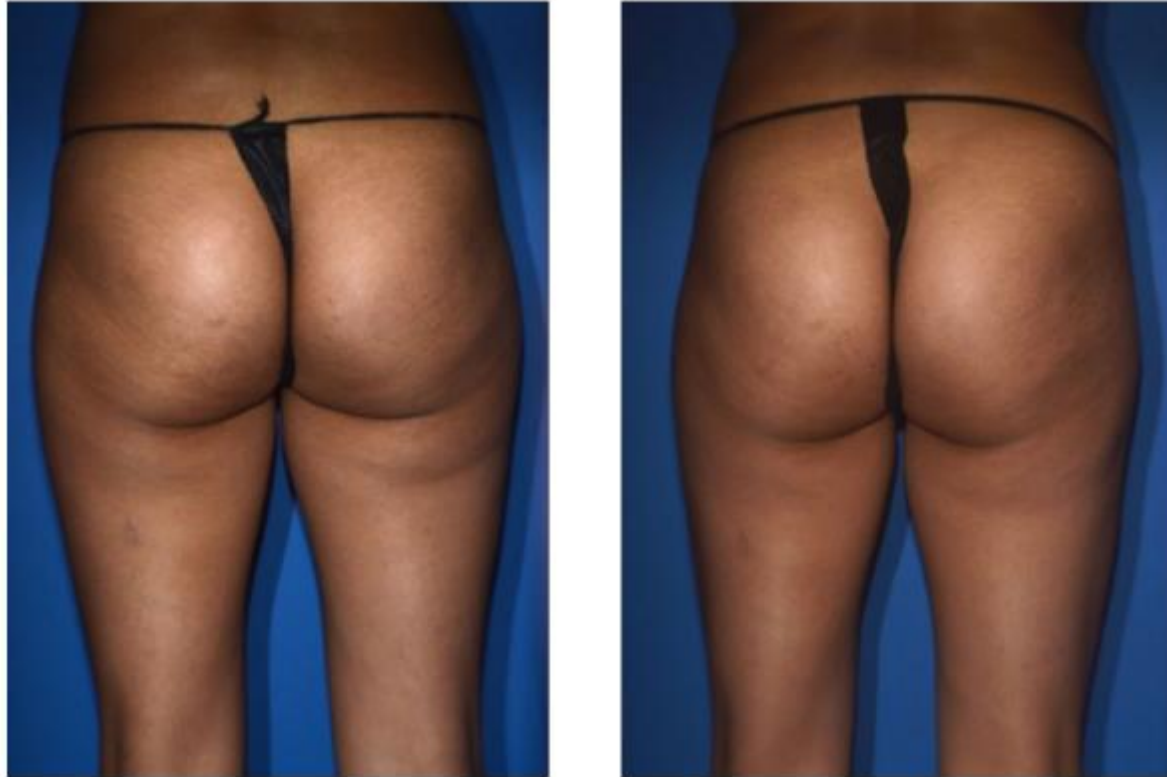
EMTONE[®]

BTLAESTHETICS.COM

SAFETY AND EFFICACY OF NOVEL TECHNOLOGY FOR CELLULITE
TREATMENT BASED ON SIMULTANEOUS APPLICATION OF MONOPOLAR
RADIOFREQUENCY AND TARGETED PRESSURE ENERGY

Mariano Busso, MD¹; Trevor M. Born, MD²

Example of patient photographs before (left) and after 4th treatment.

SAFETY AND EFFICACY OF NOVEL TECHNOLOGY FOR CELLULITE
TREATMENT BASED ON SIMULTANEOUS APPLICATION OF MONOPOLAR
RADIOFREQUENCY AND TARGETED PRESSURE ENERGY

Mariano Busso, MD¹; Trevor M. Born, MD²

BASELINE

1 MONTH FU

Example of patient photographs before (left) and after 4th treatment.

EMTONE[®]

BTLAESTHETICS.COM

REDUCTION OF ABDOMINAL SKIN LAXITY IN WOMEN POST VAGINAL DELIVERY USING THE SYNERGISTIC EMISSION OF RADIOFREQUENCY AND TARGETED PRESSURE ENERGIES.

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REDUCTION OF ABDOMINAL SKIN LAXITY IN WOMEN POST VAGINAL DELIVERY USING THE SYNERGISTIC EMISSION OF RADIOFREQUENCY AND TARGETED PRESSURE ENERGIES. Prof. Klaus Fritz M.D., Prof. Carmen Salavastru, M.D., Magdalena Gyurova M.D.

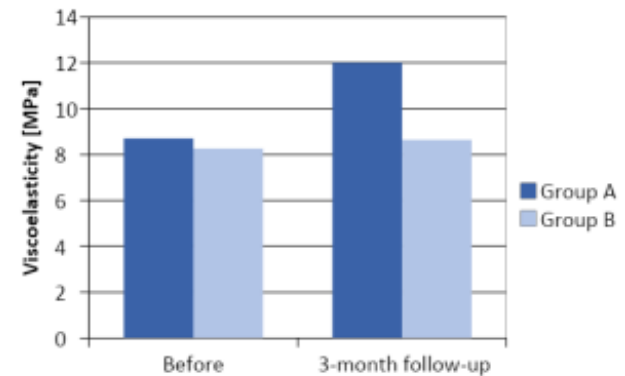
BEFORE


AFTER **BTL UNISON**

3 MONTHS AFTER 4th TREATMENT, COURTESY OF: KLAUS FRITZ, M.D.

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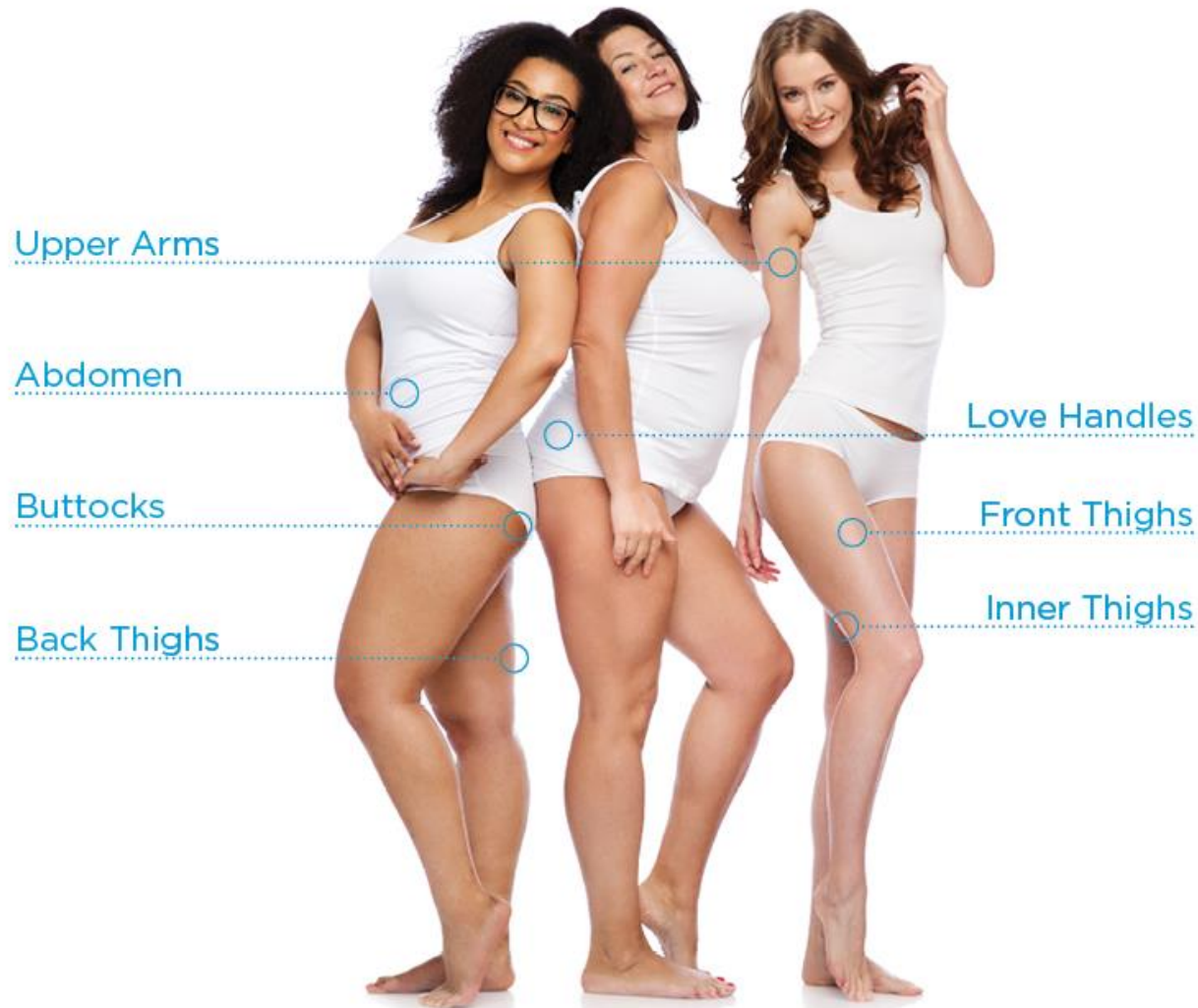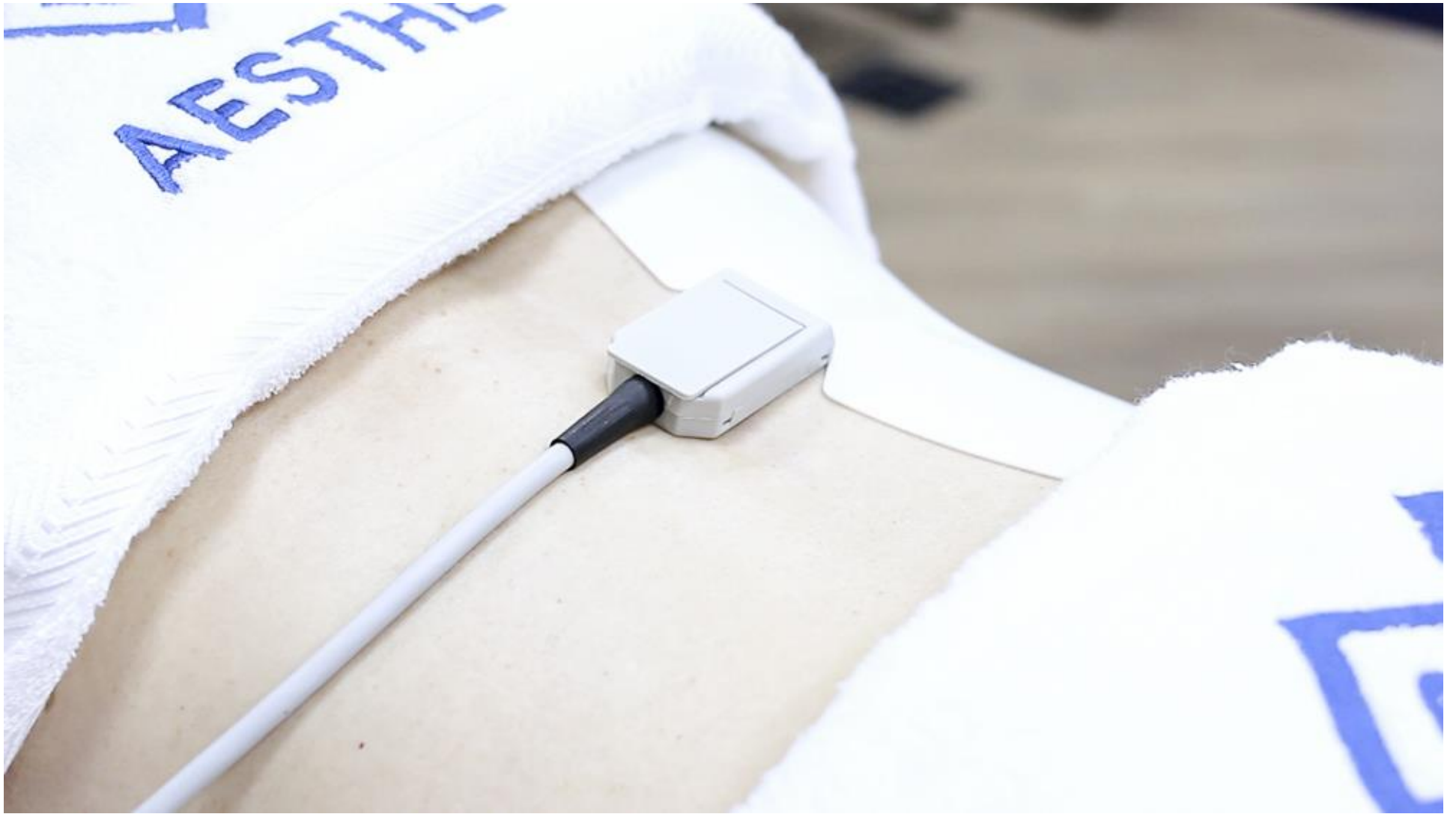
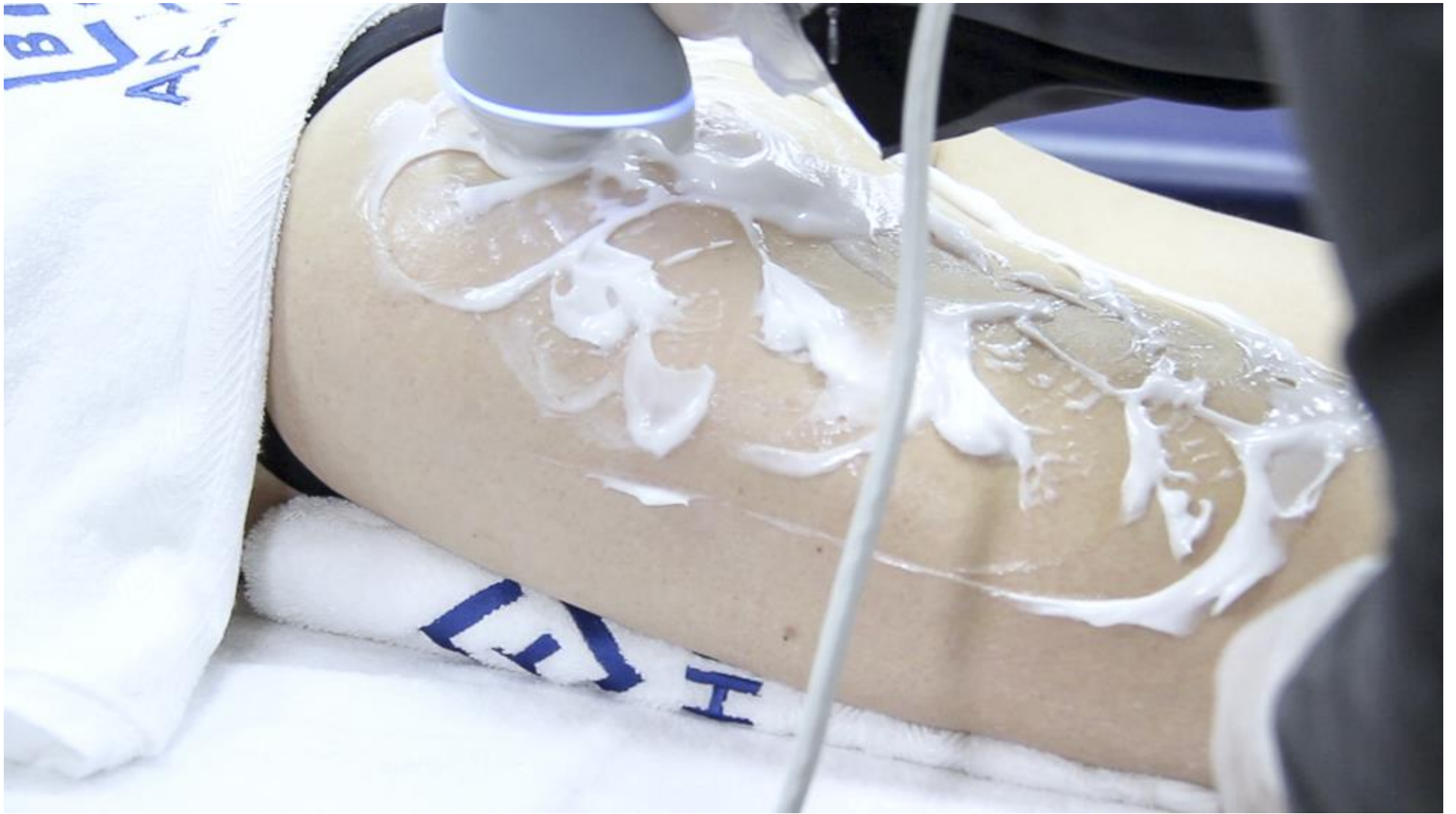
BEFORE

AFTER **EMTONE**

3 MONTHS AFTER 4th TREATMENT, COURTESY OF: BTL AESTHETICS

EMTONE[®]

BTLAESTHETICS.COM

BEFORE & AFTER

BEFORE

AFTER **EMTONE**

AFTER 4th TREATMENT, COURTESY OF: TANIA RASHID

EMTONE®

BTLAESTHETICS.COM

BEFORE & AFTER

BEFORE

AFTER **EMTONE**

3 MONTHS AFTER 4th TREATMENT, COURTESY OF: TANIA RASHID

EMTONE®

BTLAESTHETICS.COM

BEFORE & AFTER

BEFORE

AFTER **EMTONE**

AFTER 4th TREATMENT, COURTESY OF: MARC SALZMAN, M.D.

EMTONE[®]

BTLAESTHETICS.COM

BEFORE & AFTER

BEFORE

AFTER **EMTONE**

AFTER 3rd TREATMENT, COURTESY OF: GIULIA BIANCHI, M.D.

EMTONE®

BTLAESTHETICS.COM

ANTES & DEPOIS

Cortesia da BTL

Antes

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EMTONE®

BTLAESTHETICS.COM

ANTES & DEPOIS

Courtesia de Natalie Dryden, UK

Antes

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EMTONE®

BTLAESTHETICS.COM

BEFORE

AFTER **EMTONE**

AFTER 3rd TREATMENT, COURTESY OF: TOM PARMAKELLIS, M.D.

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